## Gebrauchsanleitung / Mode d'emploi / Manuale di istruzioni / Manual

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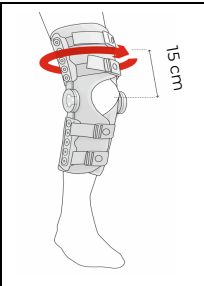
- 1 Open the knee brace, the four straps and the Velcro underneath. Place the knee brace on a stable surface. Make sure that the inside of the brace is facing upwards and that the brace is correctly aligned.
- 2 Position your injured leg so that the round recess on the back comes to rest at knee level. The knee brace closes towards the front.
- 3 Close the Velcro fasteners and then the four straps. Tighten the straps so that the brace fits properly. If necessary, the rear straps can be adjusted.
- 4 Check that the splint fits correctly and securely. Make sure that the splint is not too tight so that no pressure points occur or blood circulation is impaired.
- 5 Open the flaps of the lateral joints. To do this, push the orange latch in the direction of the arrow and lift the flap. Set the desired positions of flexion and extension limitation and close the flap again. Check that the locking flap is closed and secured correctly.

## Grössentabelle / Tableau des tailles / Tabella delle taglie / Size table

Artikelnummer Numéro d'article Numero d' articolo Article number	Grösse Taille Taglia Size		Oberschenkelumfang Circonférence des cuisses Circonferenza coscia Circumference of the thigh	
7640132380088	XS	Universal	31-38 cm	
7640132380095	S	Universal	38-45 cm	
7640132380101	M	Universal	45-53 cm	
7640132380118	L	Universal	53-60 cm	
7640132380125	XL	Universal	60-67 cm	
7640132380132	XXL	Universal	67-75 cm	
7640132380149	XXXL	Universal	75-83 cm	

# OKIN

Designed and Engineered  
in Switzerland

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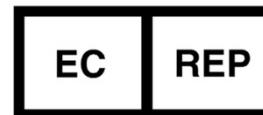
### Description

Knee orthosis for stabilisation, guidance and immobilisation of the knee due to the 4-point stabilisation principle and lateral guide rails with polycentric joints for limiting extension and flexion.

### Intended purpose

Orthosis for post- and preoperative as well as conservative and post-traumatic stabilisation, relief and immobilisation of joints and limbs.

### Indications

Preoperative, postoperative, post-traumatic, e.g. in case of ligament instability of the knee joint, meniscus injuries, stabilisation and support during interventions on the knee or chronic irritations.

### Contraindications

Lymphatic drainage disorders, also unclear soft tissue swellings remote from the applied aid, sensory or circulatory disorders in the region of the body being treated, diseases of the skin in the part of the body being treated.  
Do not use if you are allergic to any of the substances contained in this product.

### Important information

This product is intended for use on one patient only. This product is to be used by or under the direction of healthcare professionals.  
This product must not be worn over open wounds. If pain or discomfort occurs in connection with the wearing of this product, a doctor or specialist should be consulted immediately.

### Material composition

Polyester (PES) velvet, polyurethane (PUR) foam, nylon, silicone

### Care instructions

Gentle wash cycle 30 degrees. Air dry and do not use fabric softener. Remove the aluminium rails on the sides when washing.

### Storage instructions

The product should be stored in a dry place away from direct sunlight.

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