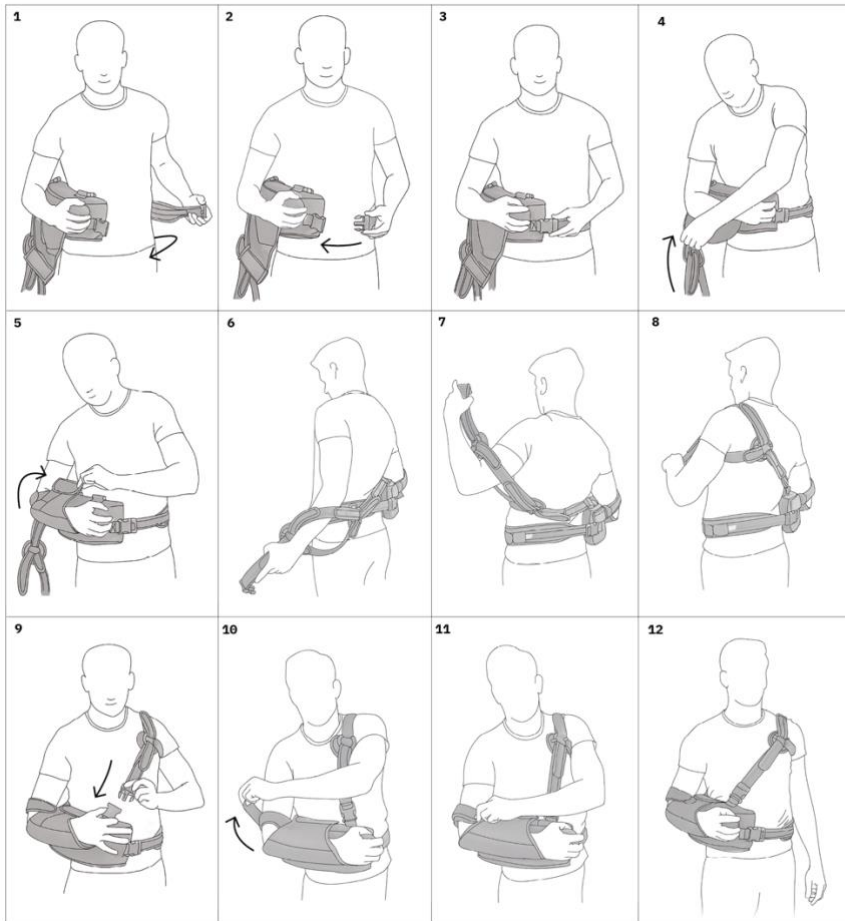
Gebrauchsanleitung / Mode d'emploi / Manuale di istruzioni / Manual

Grösstentabelle / Tableau des tailles / Tabella delle taglie / Size table

Artikelnummer Numéro d'article Numero articolo Article number	Grösse Taille Taglia Size	← ↔ →		
7640132380002	S-XL	Universal		
7640132380293	S-XL	rechts/droite/destra/right		
7640132380309	S-XL	links/gauche/sinistra/left		
7640132380187	S-XL	Universal	ER	
7640132380316	S-XL	rechts/droite/destra/right	ER	
7640132380323	S-XL	links/gauche/sinistra/left	ER	
7640132380330	XS-S	rechts/droite/destra/right		
7640132380347	XS-S	links/gauche/sinistra/left		

# OKIN

Designed and Engineered  
in Switzerland

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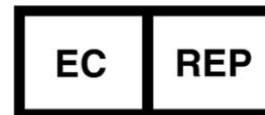
### Description

Textile orthosis for fixation of the shoulder joint and the upper and lower arm with arm strap, upper arm fixation and adjustable fixation strap for the shoulder as well as adjustable abdominal strap. In two sizes.

### Intended purpose

Orthosis for post- and preoperative as well as conservative and post-traumatic stabilisation, relief and immobilisation of joints and limbs.

### Indications

The orthoses are indicated when the corresponding joint or limb is to be immobilised, relieved or stabilised, e.g. in impingement syndrome, after injury and reconstruction of the rotator cuffs or biceps tendon, endoprosthesis implant, subacromial Defilée extension (according to Neer), subacromial bursitis, humerus fractures.

### Contraindications

Lymphatic drainage disorders, also unclear soft tissue swellings remote from the applied aid, sensory or circulatory disorders in the region of the body being treated, diseases of the skin in the part of the body being treated.

Do not use if you are allergic to any of the substances contained in this product.

### Important information

This product is intended for use on one patient only. This product is to be used by or under the direction of healthcare professionals. This product must not be worn over open wounds. If pain or discomfort occurs in connection with the wearing of this product, a doctor or specialist should be consulted immediately.

### Material composition

Polyester (PES), Nylon, Foam (PUR)

### Care instructions

Gentle wash cycle 30 degrees. Air dry and do not use fabric softener.

### Storage instructions

The product should be stored in a dry place away from direct sunlight.

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